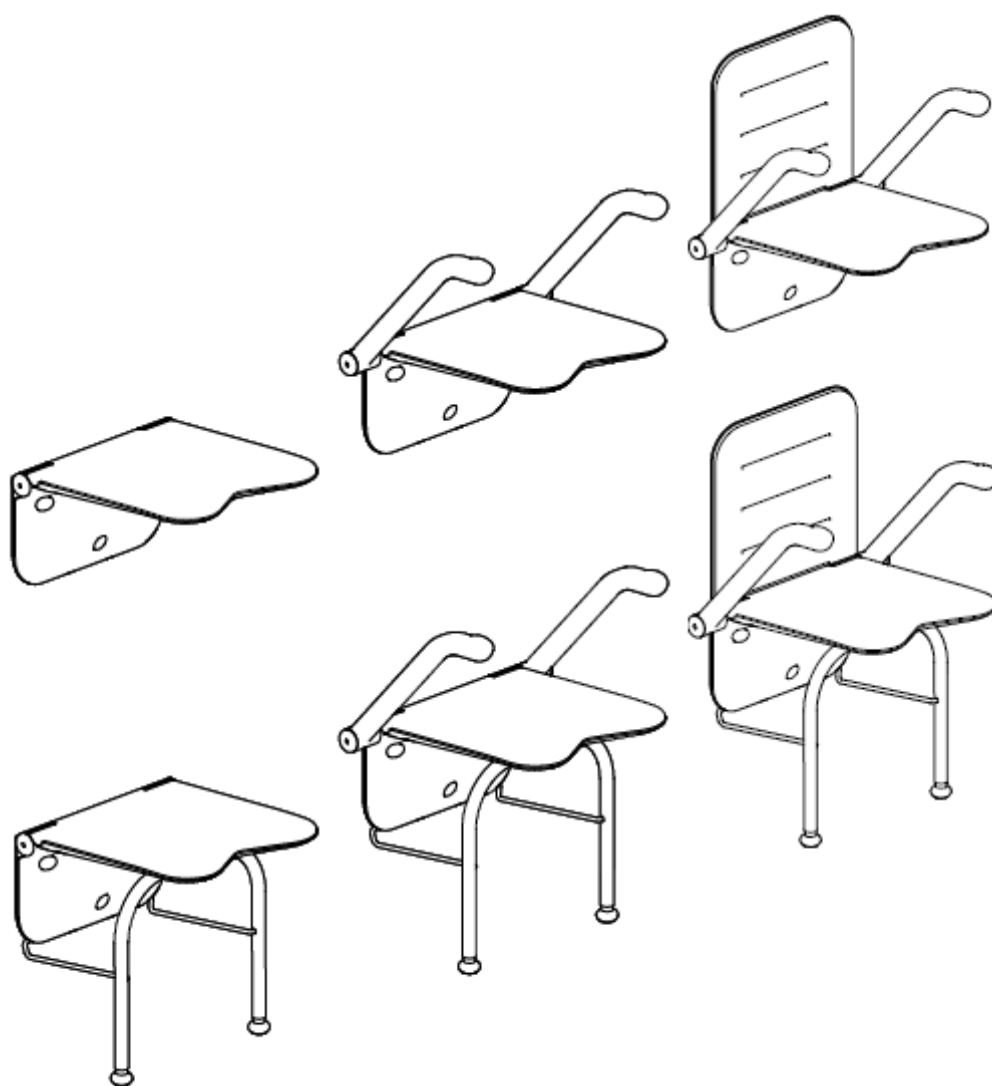
Etac Relax

 etac[®]

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2014-10-15

CE

VÝROBCE

Etac Supply Center AB
Langgatan 12
Box 203, 334 24 Anderstorp Sweden
tel +46 371 58 73 00
fax +46 371 58 73 90
www.etac.com

DISTRIBUCE A SERVIS

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English

General

Thank you for choosing a shower chair from Etac. To avoid injury during installation, handling and use, please ensure that you read this manual thoroughly and keep it. It can also be downloaded from www.etac.com. Here you will also find other documentation such as pre-sale information, pre-purchase guide and reconditioning instruction.

The Etac Relax is a wall-mounted shower seat designed as a seating and standing support for personal hygiene purposes for people with reduced mobility. The Etac Relax shower seat is used for sitting where space is restricted, and can be folded up when not in use. The Etac Relax shower seat with support legs is recommended for users weighing more than 125 kg (max. 150 kg). The Etac Relax shower seat with armrests is recommended wherever users need support to stand up. The backrest is attached to the wall above the seat and is recommended for users who need back support.

Etac Relax is intended for home use.

Etac Relax is not designed to be used in any other way than described above.

Etac Relax has been tested by test institutes and complies with the requirements set out in EN ISO 10993-1, SS EN 12182.

Etac Relax is CE marked according to Directive 93/42/EEC on medical devices.

Delivery contents

Figure A

See figure A

Installation options

May be mounted on walls with sufficient load-bearing capacity only. Please ensure that suitable fixtures are used. Each fixture must be approved for a tensile and shear load of min 4.4 kN. When mounting in non-cracked concrete C20/25 to C50/60, we recommend anchor bolt Fischer FBN II 8/30 A4 (vinyl wall covering) or Fischer FBN II 8/50 A4 (tile).

Mounting on walls with insufficient load-bearing capacity or with unsuitable fixtures can affect user safety. Mounting of the seat and assessment of the wall's load-bearing capacity must be carried out by a qualified person.

Installation

Figure B

See figure B

Usage

Figure C

See figure C

Maintenance and safety

Figure D

1. Clean/disinfect the chair with normal non-abrasive cleaning agents with a pH value of between 5-9, or with 70% disinfectant solution. Rinse and dry.
The product is otherwise maintenance-free.
2. The product can withstand cleaning in a sterilizer at 85°C for 3 minutes. This applies for reconditioning.
3. Maximum user weight (see technical data).
4. Do not stand on the shower chair!
5. Please ensure that both legs are resting against the floor when folding down the shower chair.
6. Check that the bolts are securely tightened. Defective or incorrectly installed products must not be used. In case of damage, contact your dealer.
7. Risk of crushing!
8. The armrests have each been tested and approved for half of the maximum user weight.
9. Heated chair can cause burns.

The materials used are corrosion-resistant.

The materials used are resistant to all general disinfectants.

For reconditioning and reuse, see the reconditioning instructions on www.etac.com.

Guarantee, service life

5-year guarantee against material and manufacturing defects. For terms and conditions, see www.etac.com.

The service life of the product is 10 years subject to the product being cleaned and maintained in accordance with the instructions set out in the manual and subject to proper use. The product should not be used after the expiration of the service life.

Accessories

Figure E

1. Arm support
2. Support legs
3. Back support

Label placement

Figure F

1. Product label (warnings, CE-marking)
2. Item no label (serial no, production date)

Technical data

Figure G

See figure G